

## Faceshields/PPE

Avow has faceshields available for all staff who may come in contact with patients. These protect the whole face when splashes or sprays are anticipated. Please note that these faceshields are for **one time use only and should be used in conjunction with other eyewear such as goggles**. These are available in the Lyon Center, Smith Building, or HHG should you wish to have them.

Also goggles should be cleaned/disinfected between uses by the following method:

1. Carefully wipe the outside of the goggles using a wipe or clean cloth saturated with Avow-provided disinfectant wipes such as Microkill wipes.
2. Wipe the outside of the goggles with clean water or alcohol to remove residue of the disinfectant.
3. Fully dry (air dry or use clean absorbent towels).
4. Remove gloves and perform hand hygiene.

It is important that you **NOT** use harsh chemicals **on the inside** of any eyewear. If you need to clean the inside of eyewear, you may use soap and water.

Also, this is a reminder to please discard personal protective equipment (PPE) appropriately after use.

**Do not bring used PPE to the office.**